


Narrowband UVB Home Phototherapy for Psoriasis, Vitiligo & Eczema

SolRx™ 500 Series

Hand & Foot Treatment
Spot Treatment

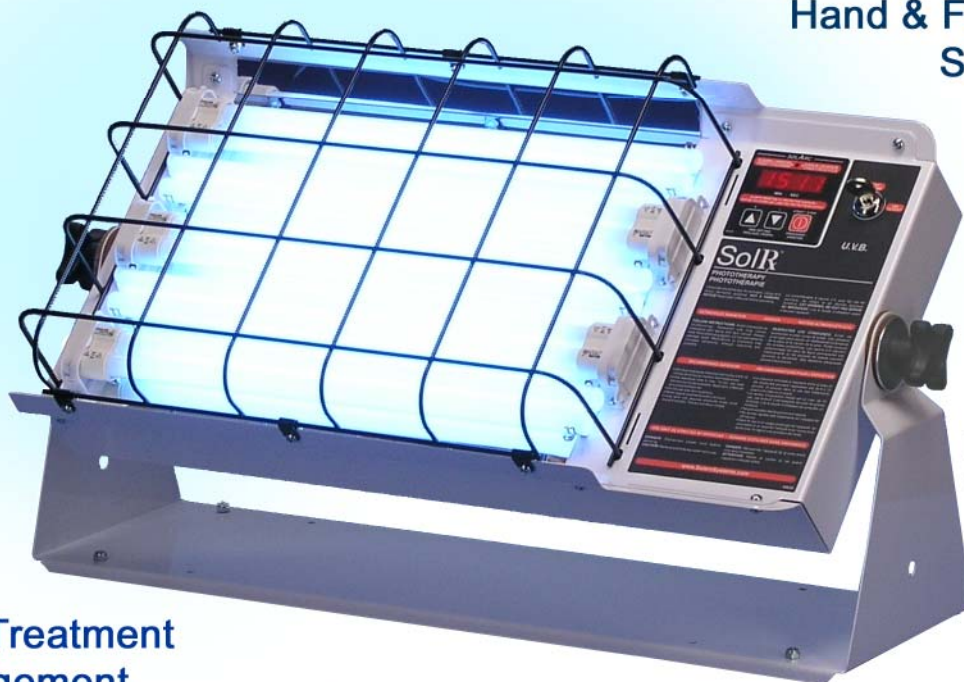


Removable
Mounting Yoke

Spot Treatment
Arrangement

Model 550UVB-NB
5 Bulbs, 180 Watts

PHILIPS

UVB-Narrowband

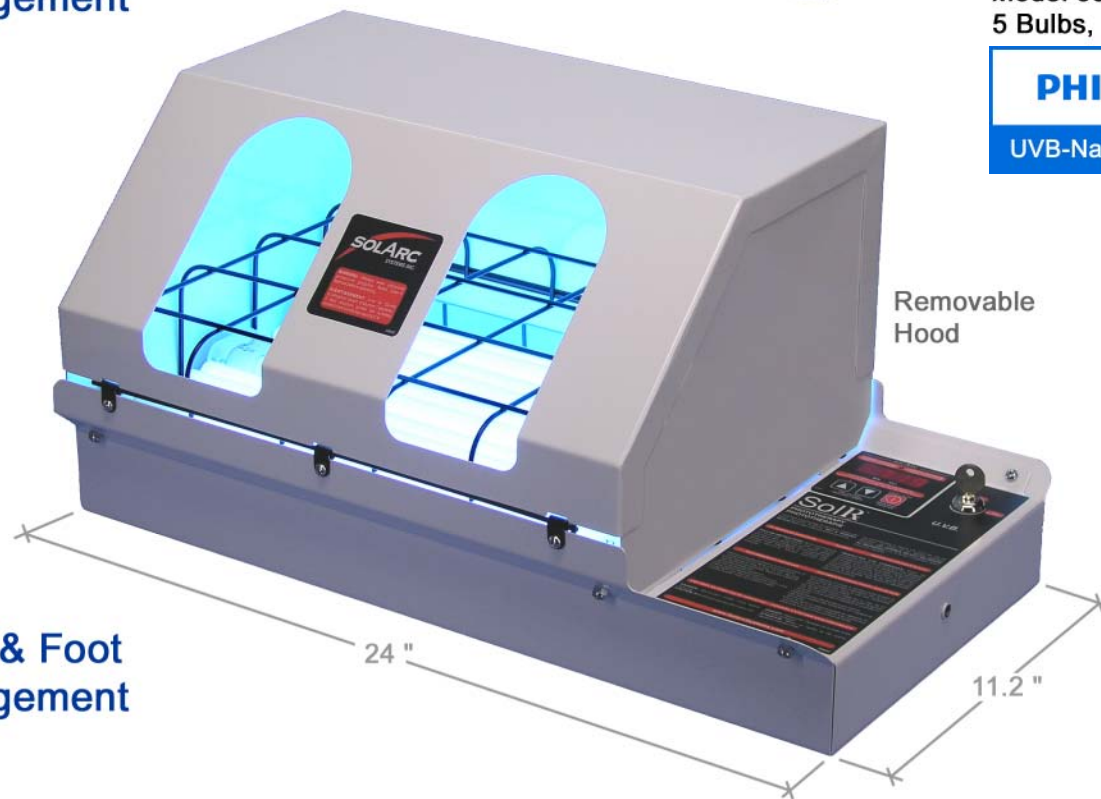

Removable
Hood

Hand & Foot
Arrangement

The **SolRx 500 Series** is a portable mid-size UVB-Narrowband device for the treatment of psoriasis, vitiligo & eczema. It has a treatment area of 18" x 13" (1.6 square feet).

This versatile unit can articulate to almost any position to provide targeted treatment of the hands, feet or any other area of the body. Powerful twin-tube compact fluorescent bulbs reduce treatment times.

See other side and SolarcSystems.com for more information.



Sol Rx 500 Series Hand/Foot & Spot Treatment




360°

The Yoke allows the device to be rotated 360° to almost any position.



In seconds you can tilt it as needed. Treatment distance is 5 to 9 inches.

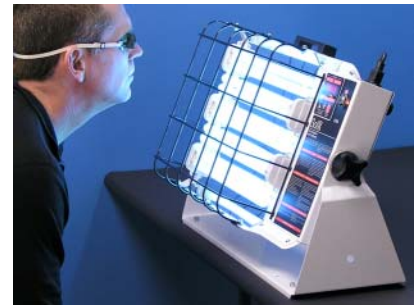

It's perfect for Narrowband-UVB treatment of the face for vitiligo.



Or put it on the floor for psoriasis treatment of the knees and leg.



For foot treatment, install the Hood and tilt. No tools are required.



Or remove the Yoke and use it as a hand/foot unit, just like at the clinic.



PHILIPS PL-L36W/01 Bulbs
SolRx™ 550UVB-NB

Powerful UVB-Narrowband twin-tube bulbs reduce treatment times.



The digital timer and switchlock are important standard safety features.



550UVB-NB
22 lbs (10 kg)

Grab it by the handle and take it anywhere !

Besides the **SolRx 500 Series'** unbeatable versatility, is its impressive light output. Modern Philips 36-watt PL-L36W/01 compact fluorescent bulbs provide considerably greater UVB-Narrowband Irradiance and shorter treatment times than competitive devices using conventional 20-watt "T12" bulbs. All of the above configurations can be obtained without the use of any tools. All units are supplied fully assembled with new bulbs, removable hood & yoke, UV protective goggles, and comprehensive User's Manual with detailed exposure guidelines for psoriasis and vitiligo. There is nothing else you need to purchase. Also available UVB-Broadband, UVA & UVA1. Please visit **SolarcSystems.com** for more info.

Dimensions (inches): 24 x 13.5 x 6.5 high (without hood or yoke)
Weight: 15 to 25 pounds (including hood & yoke)
Electrical: Uses standard 120 volt - 3 prong grounded outlet, max 1.8 amps (550CR)
Home Warranty: 4 years on device, 1 year on bulbs; plus our Arrival Guarantee

Four **UVB Narrowband** models:
520UVB-NB: 2 bulbs, 72 watts
530UVB-NB: 3 bulbs, 108 watts
550UVB-NB: 5 bulbs, 180 watts
550UVB-NB-CR (For Clinic Use)



All SolRx models are US-FDA and Health Canada compliant.

US Federal law restricts this device for sale by or on order of a Physician

Solarc Systems is ISO-13485 certified for the design & manufacturing of medical ultraviolet phototherapy equipment.



Toll Free: 1-866-813-3357 (705-739-8279)
Fax: 705-739-9684
Email: info@SolArcSystems.com
Sol ArcSystems.com

1515 Snow Valley Road, Minesing, ON L0L 1Y3

500 Series Brochure Rev5.0

